



**Non-invasive**  
technology for face  
and body beauty







**Novaclinical a business unit of Novavision Group which designs and manufactures latest-generation equipment, develops cutting-edge biotechnologies in the field of aesthetic medicine, dermatology and gynaecology.**

For over 25 years, Novaclinical professionals have been addressing the needs of a constantly evolving sector, allowing the company to retain a major role in the target market.

In addition, the great focus on investing in the research and development of cutting-edge technology makes the company a reliable partner, able to fully meet the requirements of professionals in the industry and end consumers.





---

*One multi-configurable*  
technology for several  
custom modulations

---







# *4Plus* The power of four *in your hands*

---

The device has four main features, which are reflected in its very name.

Four (and its multiples) emitters/receivers make 4Plus truly revolutionary, four technologies make its treatments really effective: from DQRF to the RSS safety system, from IoMT through to radioporation.

DQRF stands for Dynamic Quadripolar Radiofrequency, developed and patented at Novavision laboratories jointly with the best Italian universities.

Exploiting the dynamic quadripolar technology means reaching even the deepest tissues, without affecting those that are not involved in the treatment. Moreover, by using the various handpieces supplied, all face and body areas can be treated, obtaining visible results from the very first session.





# The device for the beauty of face and body



***Small Areas***  
Handpiece for small areas.



***Large Areas***  
Specifically  
designed to treat  
large areas.



***Medium Areas***  
Specifically  
designed to treat  
medium-sized areas.



***4NFR***  
Handpiece with 32  
gold-plated needles for an  
action focussing on special  
targets.



***4RFH***  
Handpiece with 384 golden  
μNeedle that can penetrate  
the stratum corneum without  
adversely affecting tissues.



## Novavision Group safeguards itself while safeguarding you

**NOVACERT** is an authentication  
system to safeguard the Novavision  
Community and the Group itself.

Each 4Plus device and component is  
equipped with a system based  
on NFC tags which allow you to  
access a webpage that certifies  
the authenticity of the product/  
component as well as its origin and  
maintenance status.





## GLUTEUS


Firming  
Modelling (lipolysis)  
Stretch Marks



## BODY

4NFR-4RFH

Scars  
Stretch Marks

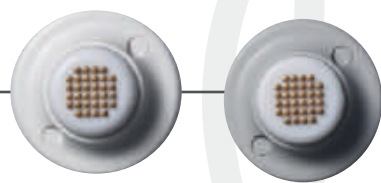

## THIGHS

Firming  
Modelling (lipolysis)  
Stretch Marks



## KNEES

Firming  
Modelling (lipolysis)  
Stretch Marks



## ARMS

Firming  
Modelling (lipolysis)  
Stretch Marks



## ABDOMEN

Firming  
Modelling (lipolysis)  
Stretch Marks



## HIPS

Firming  
Modelling (lipolysis)  
Stretch Marks



## FACE

Lifting Effect  
Wrinkles  
Face Contouring



## HANDS



Rejuvenation  
Hyperpigmentations  
Wrinkles



## FACE & NECK

4NFR-4RFH

Resurfacing  
Laxity  
Wrinkles  
Hyperpigmentations  
Acne  
Sacs

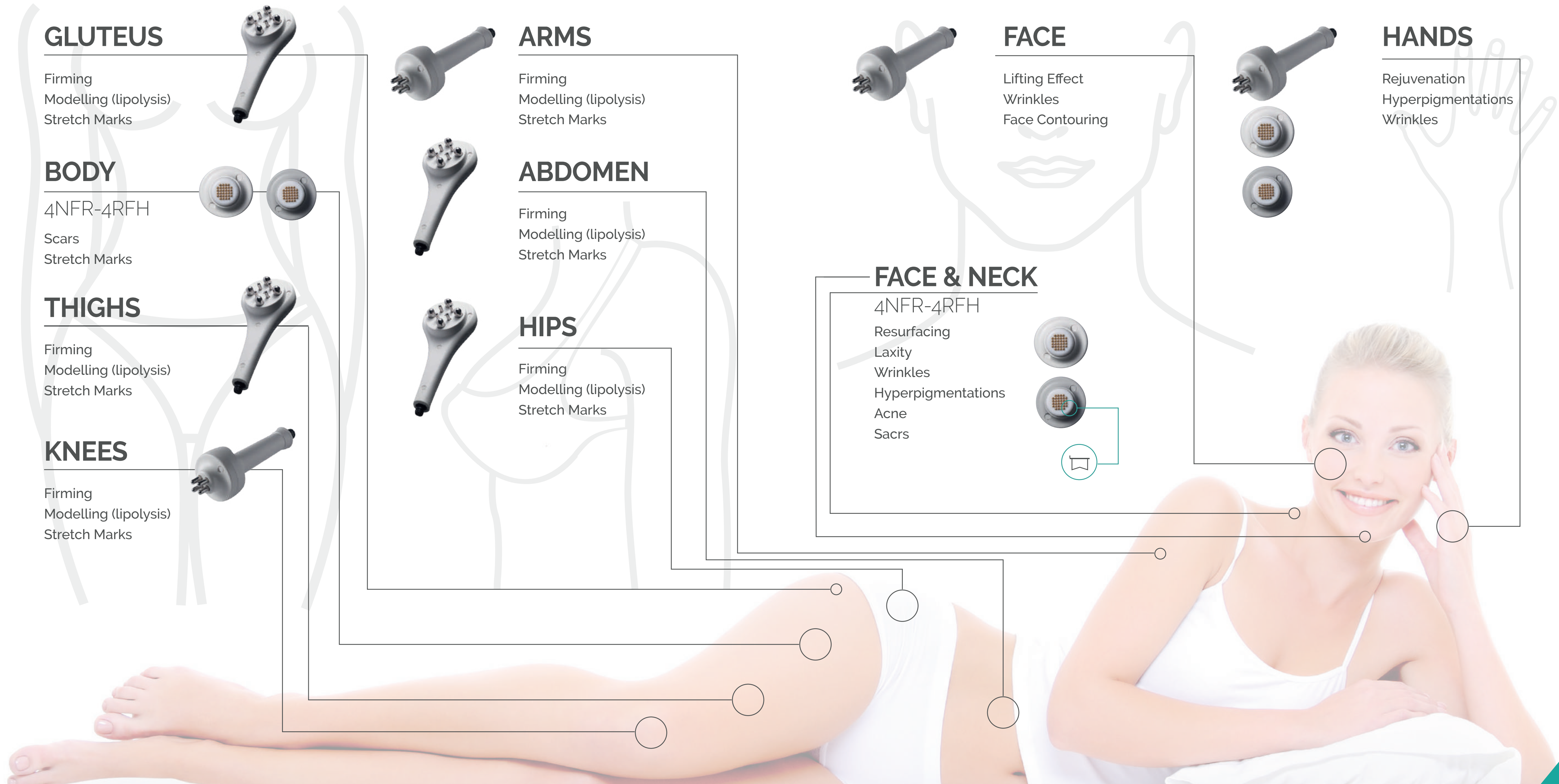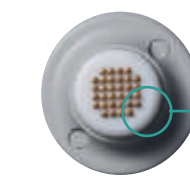



# TOTAL

*Face & Body*


Specifically designed to treat delicate areas as lip and eyes.

*area*  
**extra** *small*

The particular shape of electrodes allows a greater focus of energy on the most delicate tissues.

*New* **programs**

- Eye wrinkles
- Eye oedema
- Dark circle
- Lip Volume
- Lip Wrinkles
- Nasolabial Wrinkles
- Scars

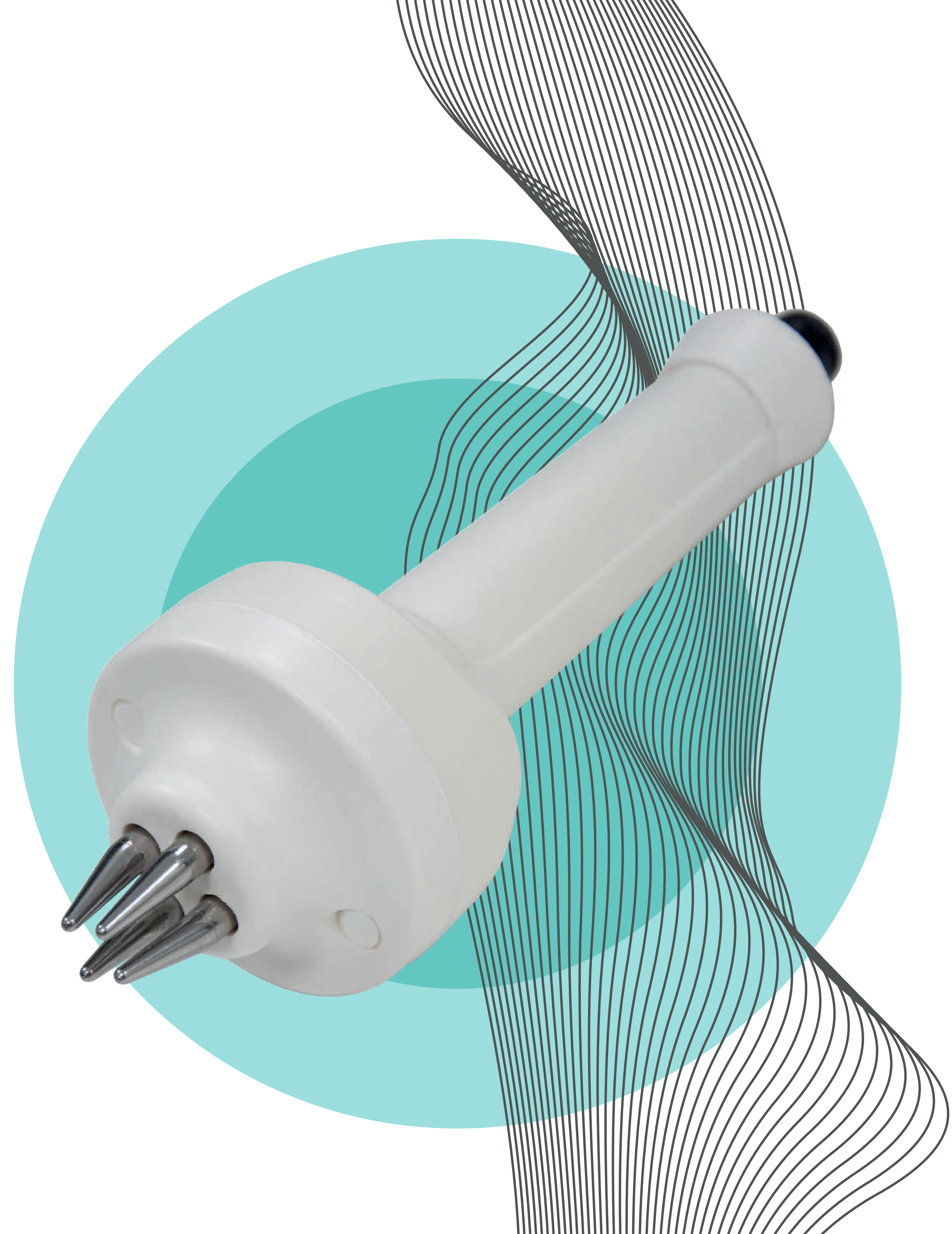









# Nova Technology

We develop highly specialised, never invasive ***technologies*** fully respecting the human body's natural physiological processes, to obtain ***effective and comfortable*** treatments.



- VRF™ — Variable Radiofrequency 
- RSS™ — Radiofrequency Safety System 
- UPR™ — Ultrapulsed Radioporation 
- IoMT™ — Internet of Medical Things 





*The tissue is treated in a natural, safe, comfortable and effective way.*

4Plus is provided with DQRF technology, owned by Novavision Group. The dynamic quadripolar radiofrequency emission focus energy on the layer of the tissue that needs treating, reducing both power needed and risks dramatically.



dynamic quadripolar radiofrequency



FACE  
NECK




ARMS



HANDS



ABDOMEN  
HIPS

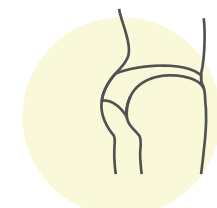

GLUTEUS



VULVA  
VAGINA




LEGS  
KNEES

## Dynamism and Configuration flexibility

Novavision engineers have managed to make the configuration DYNAMIC – the emitters turn into receivers by changing their charge within specific time spots, each time modifying the magnetic field and focusing it to obtain:

- waves that can penetrate even the deepest tissues without overheating tissues that are not involved in the treatment
- amplified common RF action in total safety
- no energy dispersion

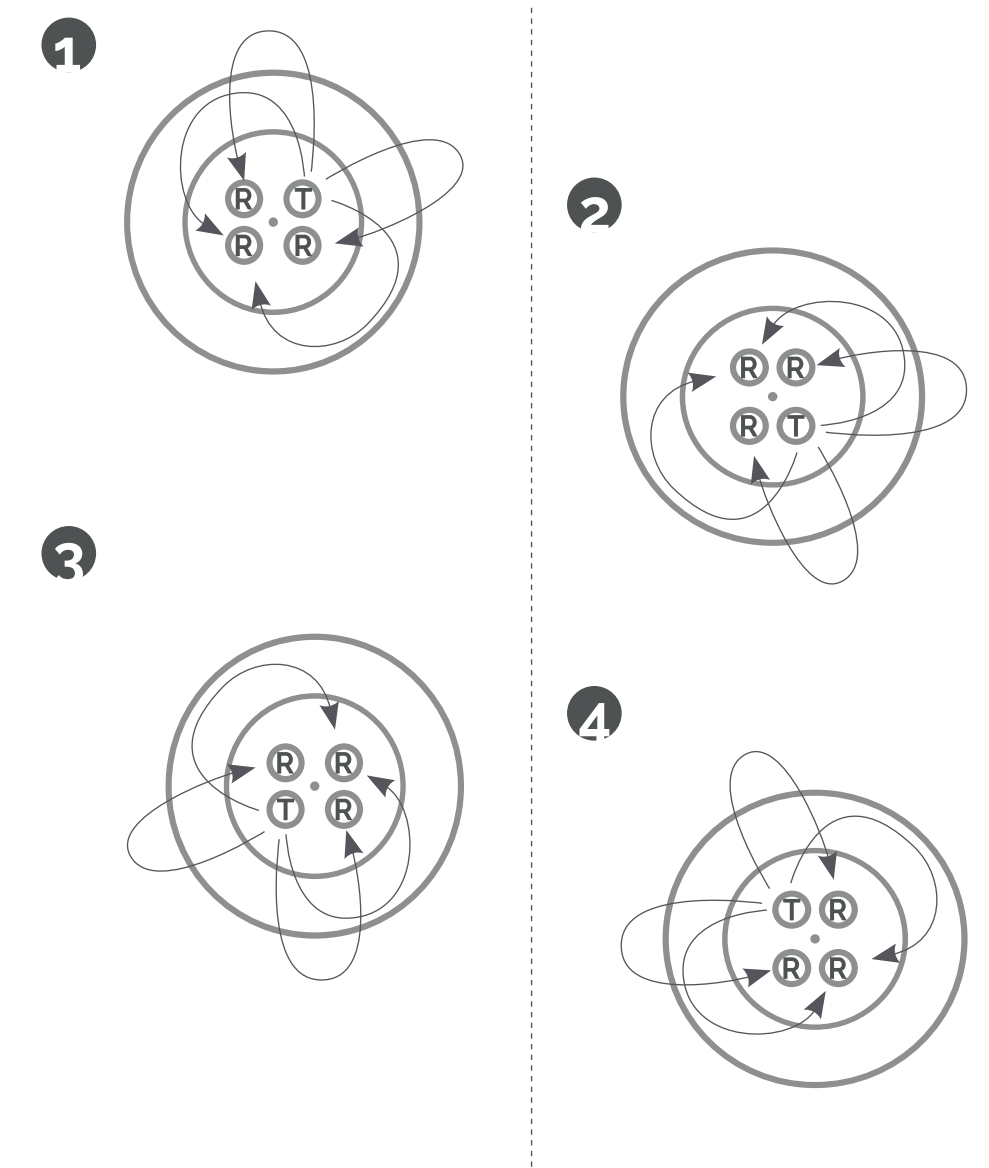






# 4PLUS Application Fields

1


SKIN RESURFACING  
EPIDERMIS

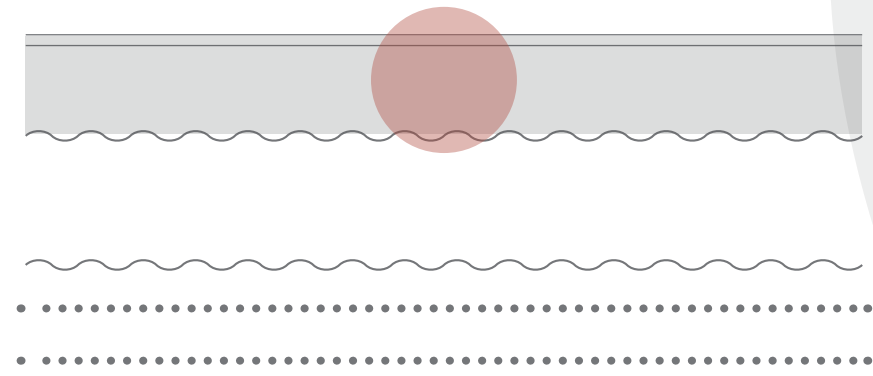

- Renewal of the skin texture
- Boosting physiological cellular turn-over

2


FIRMING  
DERMIS

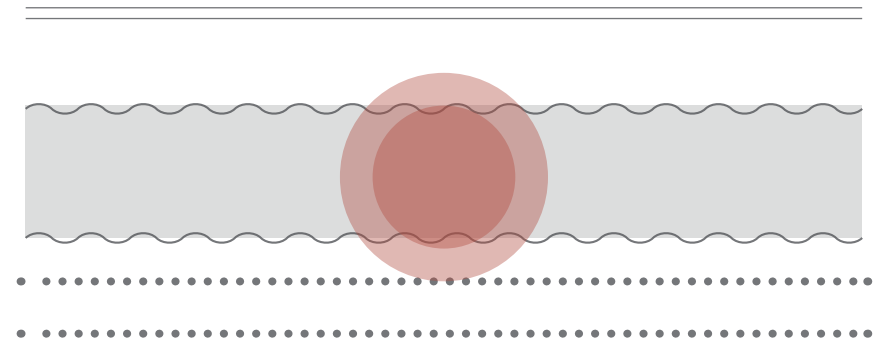

- Dermal oxygenation improvement
- Enhancement of collagen and elastin production

3

LYPOLISIS  
HYPODERMIS

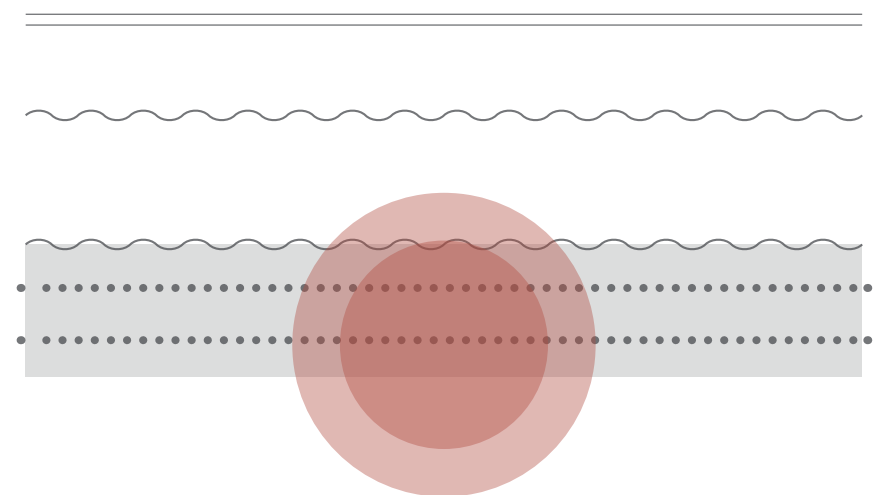

- Adipose cells volume reduction
- Resolution of fat into FFA and glycerol

 *focused endogenous heat*






dynamic quadripolar radiofrequency

*The tissue  
is treated in  
a natural,  
safe,  
comfortable  
and effective  
way.*



## VRF Variable Radiofrequency

Thanks to its preset frequency range, the exclusive VRF technology makes it possible to focus energy on the specific desired tissue layer, from the most superficial to the deeper layers. Treating dermis or hypodermis safely is now possible thanks to the combined action of VRF and DQRF technologies.

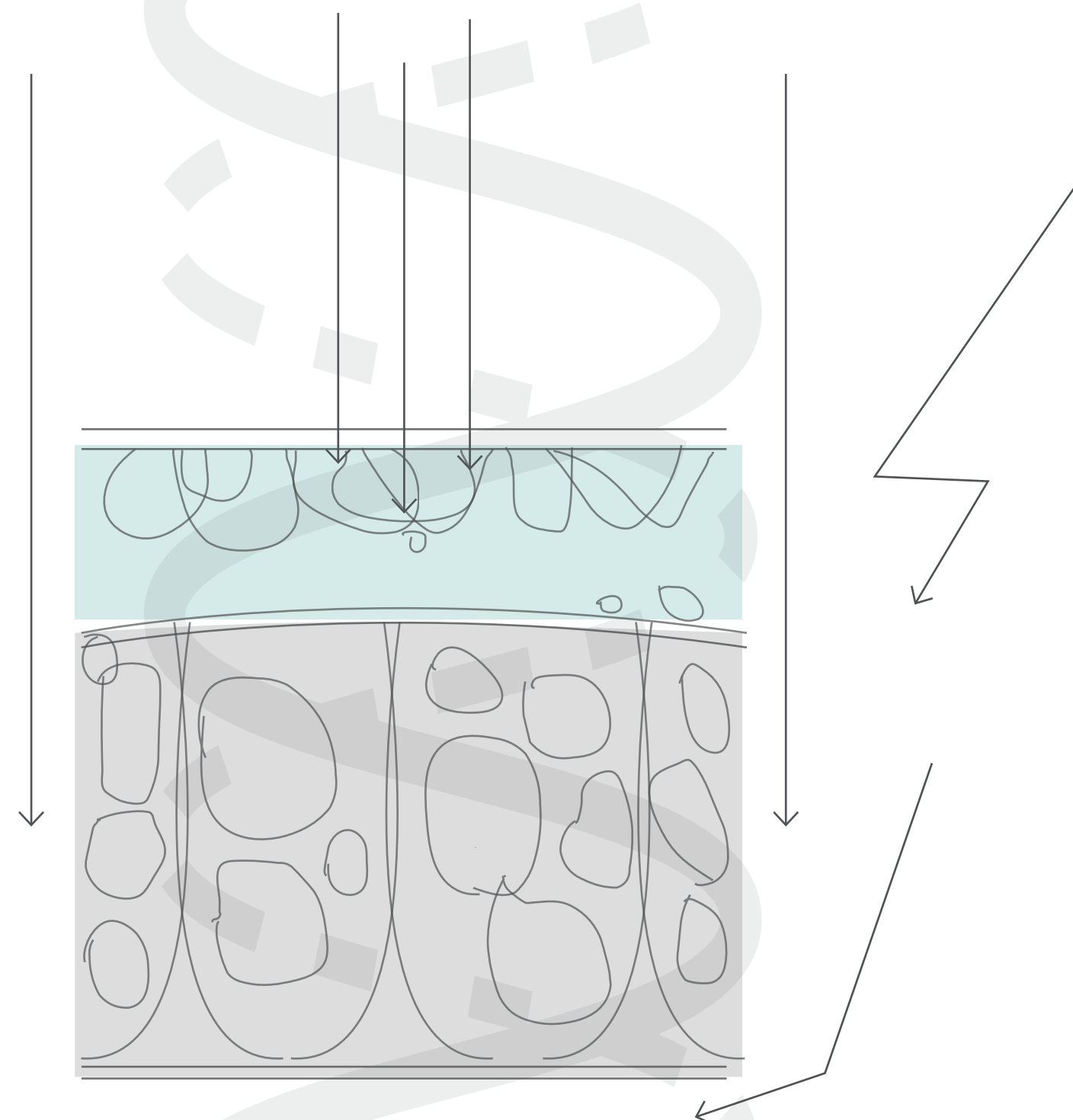





## RSS™ Radiofrequency Safety System

RSS™ is the safety system that makes 4 Plus™ treatments safe for both the patient and the operator. The Radiofrequency Safety System combines the assisting action of a movement sensor together with that of temperature detectors, both featured together in the same hand-piece.

- A system monitoring the handpiece movement ensures complete safety in operations, preventing excessive stationary position/incorrect movements and thereby avoiding harmful prolonged administration of the electrical energy onto the tissue.
- A tissue temperature detection system enables the operator to check that the effective increase in temperature is reached, while protecting the patient against overheating, as this is limited automatically by the system, which duly notifies the operator and shuts off the energy emission if necessary.

RSS technology also directly protects the patient, who will have total control of the session: thanks to Bio-Feedback, she will be able to stop the treatment by simply pressing a button, thus playing an active role in the session.


### RADIOFREQUENCY SAFETY SYSTEM

- OPERATOR MOVEMENT CONTROL
- TISSUE TEMPERATURE CONTROL
- BIO-FEEDBACK



## UPR™ Ultrapulsed Radioporation

UPR™ is the most innovative technology which, during the treatment, by using high energy micro-pulses, makes it possible to increase cell membrane permeability as well as opening aqueous channels, thus conveying the active substances.

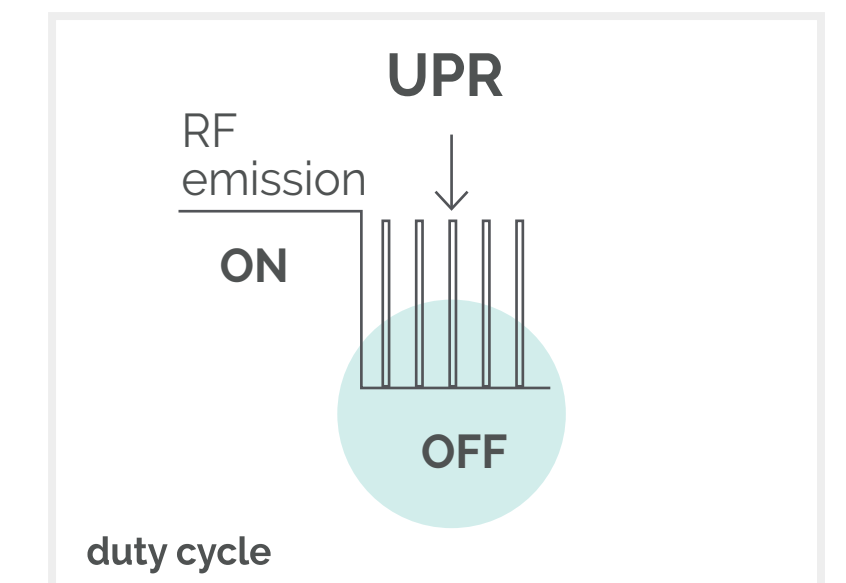





## Internet of Medical Things

---

In a rapidly changing and increasingly networked world, Novavision firmly believes in the creation and development of the IoMT™ concept. With a view to offering you a device that is always up-to-date, 4Plus is set up for continuous upgrades of the Internet of Medical Things.

This new approach makes the company available anytime and anywhere for its customers: services, remote support, training and online protocols, updates, data storage. At the same time, as a result of becoming members of the IoMT™ community, users can leverage the continuous support of the scientific board, and share experiences with colleagues around the world (customised protocols, special cases, medical history, follow-up data). The professional user can thus offer their patients unique advantages, such as access to procedure logs, wearable devices, advice, suggestions on treatments, and personal reminders.





# Fields of application

Patients increasingly require non-invasive medical aesthetic treatments to avoid surgery, anaesthesia and the related recovery times.  
4Plus™ is the innovative solution for:

- FACE LIFTING
- BODY RESHAPING
- FACE CONTOURING
- BODY FIRMING
- RESURFACING
- TARGETED ANTI-SCARRING ACTION
- STRETCH MARK REDUCTION

## Targeted solutions

DQRF™ treatments involve a wide target of young to adult patients, to address the various problems of different age groups.

	20 years	30 years	40 years	50 years	>60 years
scarring					
hand rejuvenation					
cellulitis					
stretch marks					
resurfacing					
targeted fat reduction					
reshaped figure					
wrinkle reduction					
skin tone and elasticity					
acne scars					
discolouration					





---

Why is 4Plus™  
the *right solution*  
for your  
patients?

---

### **Zero downtime with 100% comfort**

DQRF treatments do not involve recovery times, which means no change to one's daily habits.

### **“Lunchbreak” sessions with visible results**

Short sessions that can be easily scheduled accommodate both the patient and the operator's needs and provide immediately perceptible results.

### **No pain treatments**

Thanks to painless procedures and with total comfort for the patient, no anaesthesia or cooling systems are required.

### **NON ablative solution**

DQRF technology stimulates physiological cell regeneration without damaging the treated areas.







---

Why is 4Plus™  
the *right solution*  
for your  
office?

---

## **Simple procedures effective protocols**

Novavision proprietary technologies assure ease of use.

## **No energy dispersion**

Thanks to the DQRF technology embedded in the device, the target area can be treated with lower energy dispersion on the tissues that are not involved in the treatment.

## **Safe Operator**

4Plus protects the patient as well as the operator, who can perform treatments throughout the day, with no time limits and in total safety.

## **Training in equipment use**

Novavision provides constantly updated training material, multimedia support on how to properly use the equipment, and ongoing training with experienced professionals in the field.

## **Real-time support**

Thanks to IoMT technology, each device is directly connected to the parent company, allowing the Novavision team to provide real-time remote support.





Novavision Academy is Novavision Group's team dedicated to training and scientific courses for the group's community.

Every year, a team of professionals defines a schedule of refresher courses on topics such as electrophysiology, applied pathophysiology, as well as specific training in the use of the device and protocol customisation. The courses will be held on our hq or through online webinars.

## WHAT IS IT?

The Novavision Group offers its community the opportunity to take part in scientific courses on prominent topics in the medical field and on the use of the device.

## HOW DOES IT WORK?

Every year the team defines a schedule of meetings that can be viewed on [novavision.net](https://novavision.net). To learn more, just contact our offices and book a position for the chosen course.

## WHAT DOES IT OFFER?

A team of experts will train you in the various topics, supporting you with multimedia material and/or real time demos, and you will receive an official Novavision certification.





# Total made in Italy


**100% made in Italy** means Italian design, Italian materials, Italian data sheets and the guarantee of using the highest quality products.

Everything you need – from consumables to spare parts – will be shipped directly from our laboratories, which means immediate and effective support.



## Roi


40

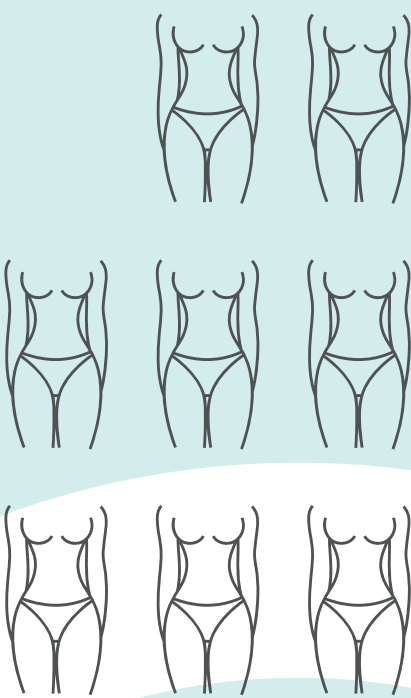

With just **40\* treatments** you can fully recoup the investment!

\*Result obtained from the average of the number of sessions per treatment and the related price, based on data received from the operators that already use the device.



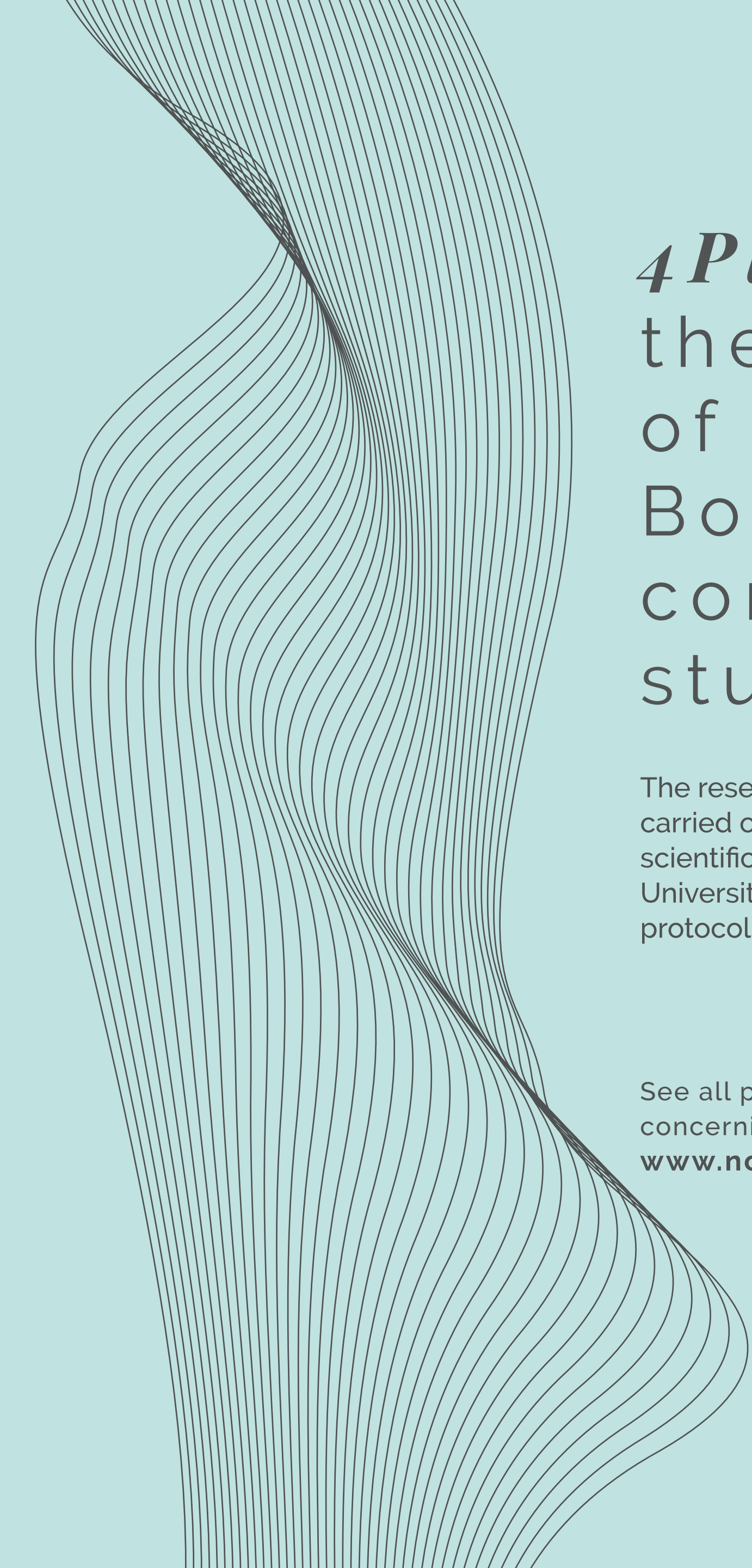

# *4Plus* is the result of the Scientific Board's continuous study efforts

The research that is steadily carried out by **Novavision Group's** scientific board jointly with innovative Universities validates 4Plus™ treatment protocols and applications.

See all publications  
concerning 4Plus™ at  
**[www.novaclinical.it](http://www.novaclinical.it)**

## Our users... are our influencers.

### ***Rosy***

*55 years old*

"I have a relaxed, bright face at last; I got immediately visible results after a single 4Plus session. I had tried various radio frequency treatments but dynamic quadripolarity, featuring no energy dispersion during therapy, made the difference. My doctor recommended a 6-treatment cycle for my face. Impressive, visible results".

### ***Giulia***

*45 years old*

"I had annoying crow's feet around my eyes and sagging eyebrows. Thanks to a cycle of treatments with the 4PLUS fractional handpiece, my wrinkles were alleviated and I was able to avoid more invasive and ablative operations".

### ***Massimo***

*43 years old*

"My annoying belly fat was sapping my confidence – scalpel fear, what could I do? They recommended 4Plus that, with VRF, effectively acts on fat. A pleasant, painless, safe and effective treatment. I solved the problem in just 8 treatments".

4Plus™, DQRF™, RSS™, UPR™, VRF™ are trademarks of Novavision Group S.p.A.





**Novaclinical a business unit of Novavision Group S.p.A.**  
Via dei Guasti, 19/29 - 20826 Misinto (MB) Milan - Italy  
☎ +39 02 967 202 40 | ✉ info@novavision.net | 🌐 www.novavision.net  
**www.novaclinical.it - info@novaclinical.it**

DC.PB.0008.R0  
In order to improve its products, Novavision Group reserves the right to change the technical characteristics of its devices without notice. All rights reserved.

