



CASE HISTORIES

CONDITION: Skin Dyschromias



SESSIONS 2
1 TREAT/WEEK

TREATMENT: Face Lifting + Face hyperpigmentation

HANDPIECE: Small + 4RFH™

SESSION DURATION: Face Lifting 15 min. + Face hyperpigm. 2 passages

POWER: 14 % + Face hyperpigmentation 10%

QUADRIPOLE RADIOFREQUENCY

CONDITION: Skin Laxity



SESSIONS 1
1 TREAT/WEEK

TREATMENT: Face Lifting

HANDPIECE: Small

SESSION DURATION: Face Lifting 15 min.

POWER: 15%



QUADRIPOLE RADIOFREQUENCY

CONDITION: Photoaging



SESSIONS 6
1 TREAT/WEEK

TREATMENT: Face Lifting + Antiage

HANDPIECE: Small + 4RFH™

SESSION DURATION: 20 min. + 2 Passages

POWER: Face Lifting 15% + Antiage 8%



QUADRIPOlar RADIOFREQUENCY

CONDITION: Skin Dyschromias & Laxity



SESSIONS 3
1 TREAT/WEEK

TREATMENT: Face Lifting + Dermopeeling

HANDPIECE: Small + 4RFH™

SESSION DURATION: 20 min. + 2 Passages

POWER: Face Lifting 15% + Dermopeeling 10%

QUADRIPOlar RADIOFREQUENCY



CONDITION: Neck Laxity



SESSIONS 6
1 TREAT/WEEK

TREATMENT: Lifting Décolleté + Antiage
HANDPIECE: Small + 4RFH™
SESSION DURATION: 15 min. + 2 Passages
POWER: Lifting Décolleté 15%+ Antiage 10%

QUADRIPOlar RADIOFREQUENCY



CONDITION: Wrinkles



SESSIONS 2
1 TREAT/WEEK

TREATMENT: 1 session Strong Dermopeeling +
2 session Face Lifting

HANDPIECE: 4NFR™ + Small

SESSION DURATION: Antiage 3 Passages + 15 min.

POWER: 14% + Strong Dermopeeling 16%

QUADRIPOlar RADIOFREQUENCY



CONDITION: Lips Wrinkles



SESSIONS: 3

TREATMENT: Face Dermopeeling

HANDPIECE: 4RFH™

SESSION DURATION: 3 passages

POWER: 8-12%



QUADRIPOlar RADIOFREQUENCY

CONDITION: Eyes Wrinkles



SESSIONS 3
1 TREAT/WEEK

TREATMENT: Eye Wrinkles + Dark Circle

HANDPIECE: Extra Small + UPR rejuvenation

SESSION DURATION: Eye Wrinkles 5 min. + Dark Circle 5 min.

POWER: Eye Wrinkles 8% + Dark Circle 8%



QUADRIPOlar RADIOFREQUENCY

CONDITION: Wrinkles



SESSIONS 3
1 TREAT/WEEK

TREATMENT: Face Lifting + Eye Wrinkles

HANDPIECE: Small + Extra Small

SESSION DURATION: Face Lifting 15 min. + Eye Wrinkles 5 min.

POWER: Face Lifting 20% + Eye Wrinkles 8%

QUADRIPOlar RADIOFREQUENCY



CONDITION: Wrinkles



SESSIONS 3
1 TREAT/WEEK

TREATMENT: Face Lifting + Eye Wrinkles

HANDPIECE: Small + Extra Small + UPR

SESSION DURATION: Face Lifting 20 min. + Eye Wrinkles 5 min.

POWER: Face Lifting 20% + Eye Wrinkles 8%

QUADRIPOLE RADI OFREQUENCY



CONDITION: Nasolabial Wrinkles



SESSIONS 4
1 TREAT/WEEK

TREATMENT: Face Lifting + Nasolabial Wrinkles

HANDPIECE: Small + Extra Small

SESSION DURATION: Face Lifting 15 min. + Nasolabial Wrinkles 5 min.

POWER: Face Lifting 14% + Nasolabial Wrinkles 10%



QUADRIPOLE RADIOFREQUENCY